



Use the neckstrap

Dropping Moyo can damage the device.

DO

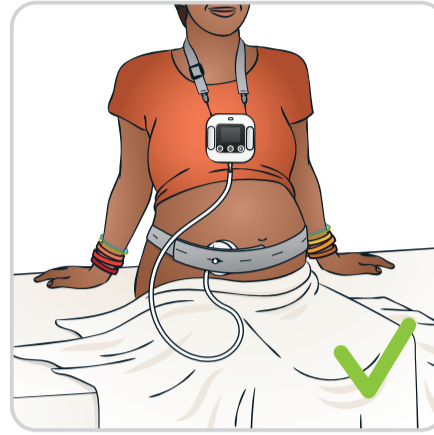

DON'T

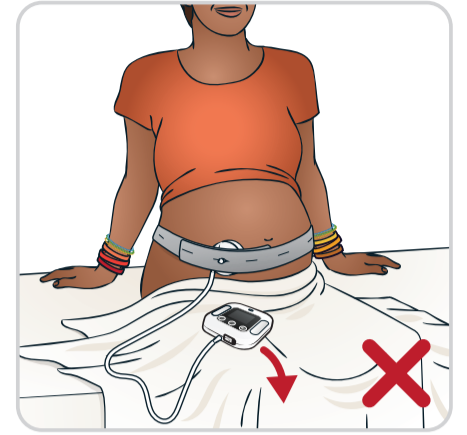


Always close the USB lid

Fluids can damage the electronics inside Moyo.





Protect the cable from tight bending

Moyo cable can break from excessive bending.

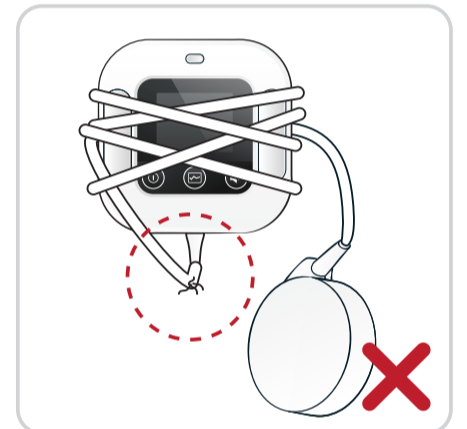


Keep the area around Moyo tidy

Due to a magnetic force in Moyo, metal objects like staples can penetrate and damage the loudspeaker.

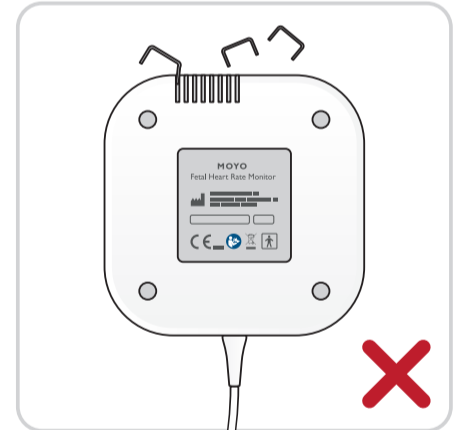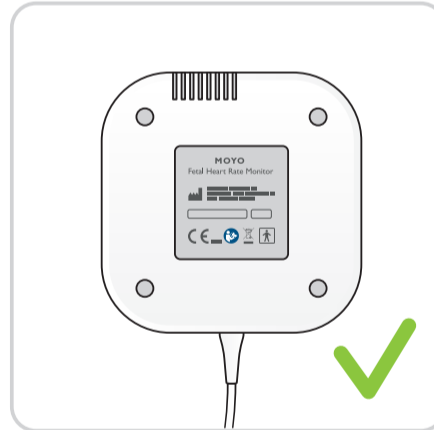

Clean using a cloth dampened with soapy water (not dripping wet)

Fluids can damage the electronics inside Moyo.

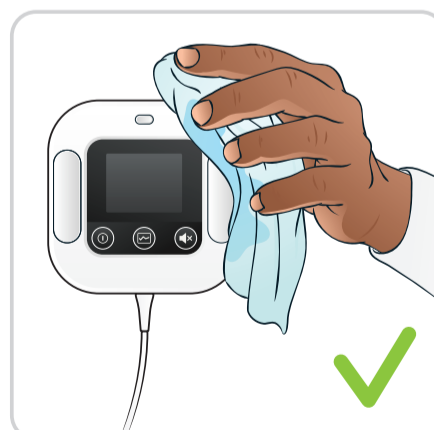

Only use 70% ethanol for disinfection (dampened cloth or spray)

Chlorine can damage Moyo.

