

# IMPORTANT! KEEP FOR FUTURE REFERENCE

Read all the instructions before using CarePlus

## CarePlus

Helping Babies Grow



Maximum weight of baby: 2.5 kg

Cat. no. LGH-950-00033

[www.laerdalglobalhealth.com](http://www.laerdalglobalhealth.com)



**Laerdal**


helping save lives

## Instructions for use

- 1 Tie securely at waist before placing the baby.



- 2 Position the baby in KMC posture. Place a soft cloth in between the baby and CarePlus. Wear wrap over the baby.

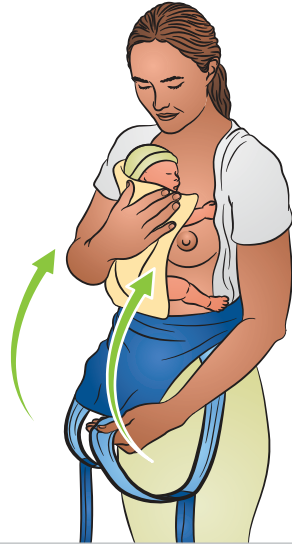

- 3 Support the baby with one hand. Cross the straps on the back with the other hand.



- 4 Bring straps to front and tie securely. Ensure the baby is snugly fit inside. Adjust CarePlus to support the neck.




## Intended use

Intended for continuous Kangaroo Mother Care (KMC) for a newborn weighing up to 2500 gram. Recommended for use after guidance from trained health worker.

## KMC Posture

While using CarePlus, ensure that the newborn has good skin-to-skin contact. The newborn's head should be well supported and visible for clear airway.

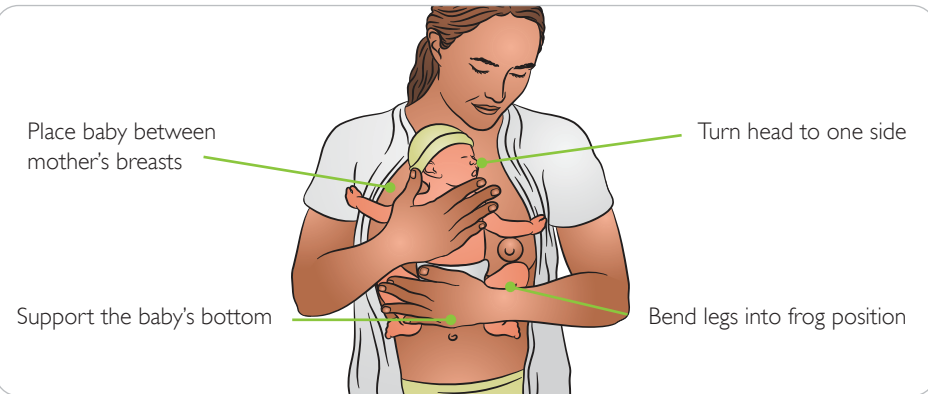

## Quality KMC


- Maintain proper KMC posture.
- Provide continuous KMC for as much time as possible.
- Frequent breastfeeding as recommended by the health worker.

## Adjustment for smaller babies

For smaller babies, fold from the bottom edge to adjust the height of CarePlus



## Recommended knot

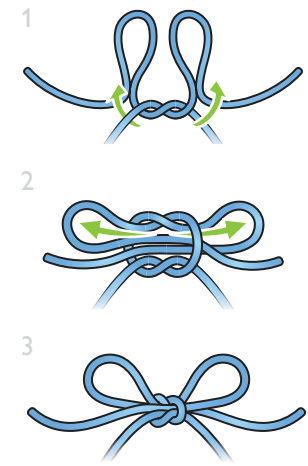

## Attention

- Protect the baby from hazards in the domestic environment, i.e. heat sources, spilling of hot drinks.
- Do not use CarePlus with more than 2 children or for more than 2.5 kg weight.
- Your movement and the child's movement may affect your balance.
- Take care when bending or leaning forwards or sideways.
- The sling is not suitable for use during sporting activities e.g. running, cycling, swimming and skiing.
- There is increased risk of your child falling out of the sling as it becomes more active.
- Regularly inspect the sling for any signs of wear and damage.
- Keep this sling away from children when it is not in use.

## WARNINGS:

- *Constantly monitor your child and ensure the mouth and nose are unobstructed.*
- *For pre-term, low birthweight babies and children with medical conditions, seek advice from a health professional before using this product.*
- *Ensure your child's chin is not resting on its chest as breathing may be restricted which could lead to suffocation.*
- *To prevent hazards from falling ensure that your child is securely positioned in the sling.*

## To clean

1. Wash in clean water with mild detergent.
2. Rinse in clean water to remove any detergent.
3. Dry completely before use.

© 2017 Laerdal Medical AS. All rights reserved.  
Laerdal® is a registered trademark of Laerdal Medical AS.

Manufactured in China for: Laerdal Medical AS  
P.O. Box 377, Tanke Svilandsgate 30, 4002 Stavanger, Norway  
T: (+47) 51 51 17 00

Printed in China.

20-12253 Rev A

